vario **DIGITAL** FHD

16591

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Operating instructions **EN**

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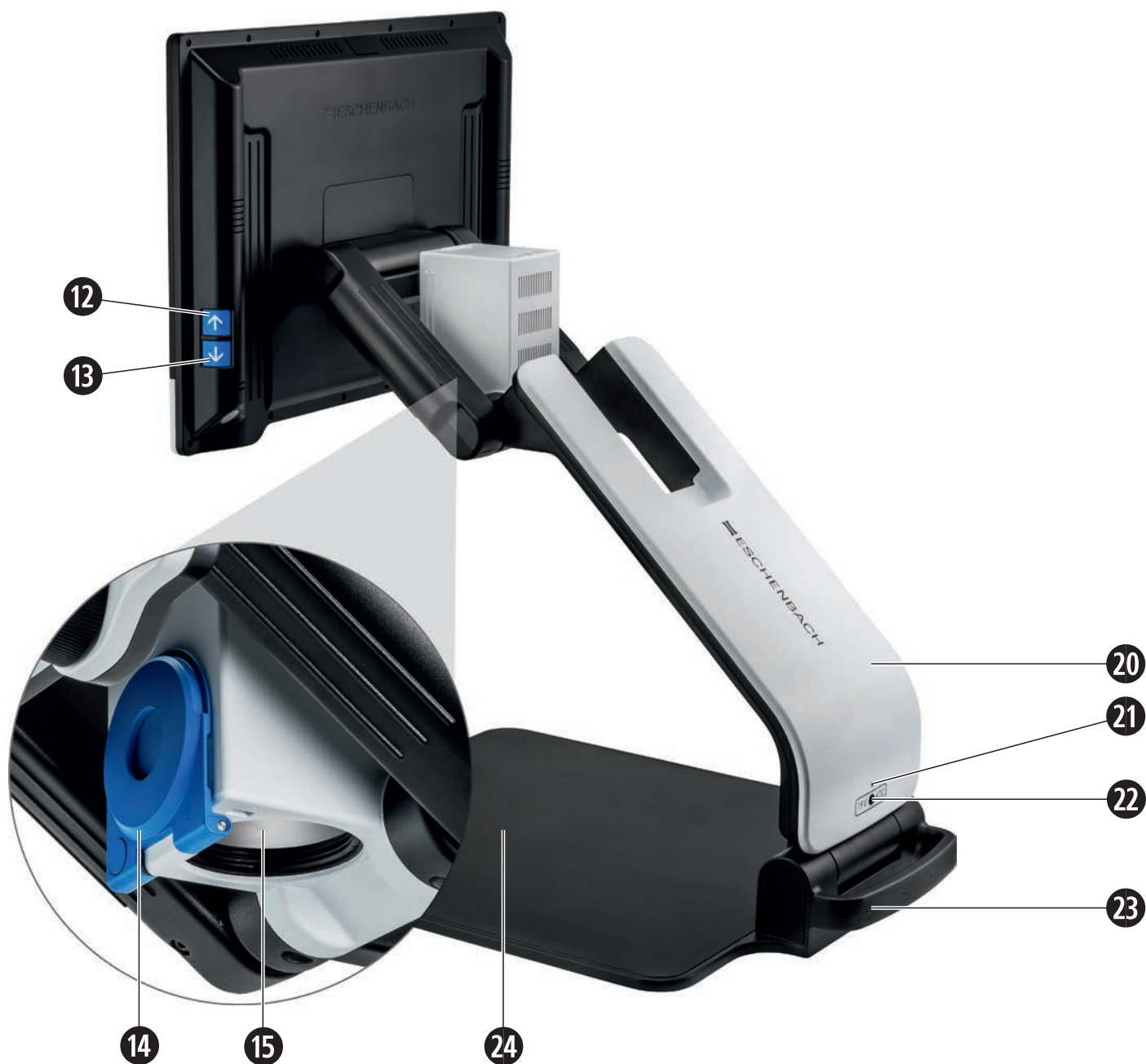
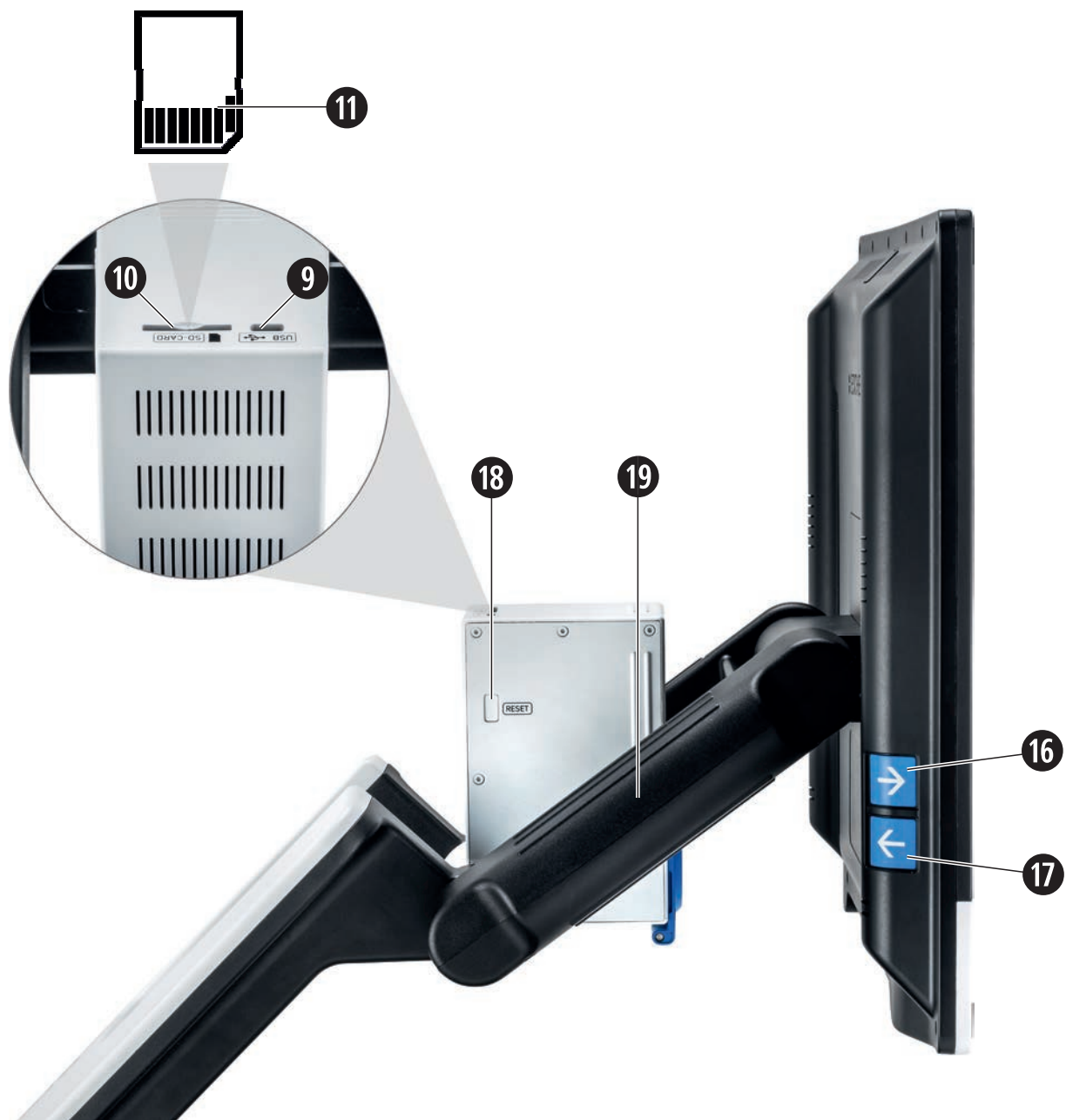
Manual de instrucciones **ES**

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 **ESCHENBACH**

Deutsch.....	1
English.....	17
Français	33
Italiano	49
Español.....	65
Nederlands	81

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Dear Customer,

Congratulations on the purchase of your Eschenbach vario DIGITAL FHD.

As market leader in magnifying glasses and reading aids we at Eschenbach can look back on many years of experience, which is reflected in the quality, handling and design of our products.

EN

Firmware version

These instructions are based on firmware version "VD16_fw_1.0".

Intended use

This electronic reading device is exclusively intended for magnification and reading or viewing of texts and images. To guarantee correct functioning, the screen reader must be set up as described.

The product is operated in a domestic environment (also includes nursing homes, workplaces, schools and other educational facilities).

Before first use

Please read these operating instructions thoroughly before using the device for the first time. This is the only way to get to know all the features available for the device.

Package contents

- ▶ vario DIGITAL FHD
- ▶ Power adapter
- ▶ Power plug adapter for EU, GB, US, AU
- ▶ USB cable (Type C)
- ▶ Microfiber cloth
- ▶ Operating instructions

Safety information

EN

- ▶ **Risk of damage to the eye and risk of injury: Never look into the light sources on the underside of the device.**
- ▶ **Never allow children to handle electric devices without supervision.**
- ▶ **Risk of suffocation and risk of accidental swallowing: Keep packaging, films and packaging parts away from children.**
- ▶ **Risk of pinching: ensure that your fingers do not get pinched during unfolding and folding of the device, and also during unfolding and folding of the camera on it.**
- ▶ **The entire device must stand on a stable surface.**
- ▶ **Take care not to leave any objects on the base when you want to fold the device together. This can result in damage to the display.**
- ▶ **Protect the device against impact or shock. Handle the device with care.**
- ▶ **Never subject the device to humidity or moisture.**
- ▶ **Protect the device from excessive heat and open flames. Never place the device on radiators or in direct sunlight.**
- ▶ **Do not store the device in locations with excessive heat, for example in a parked car.**
- ▶ **The light source contained in the device may only be replaced by the manufacturer, its customer service department or another qualified person.**
- ▶ **Use only the supplied power adapter and country-specific power plug adapter.**
- ▶ **Check the device, the power adapter and its cable for visible damage. If there is any damage, disconnect the power adapter from the power supply immediately. Have the product checked by a qualified technician before you use it again.**

- ▶ **Route the power cord so that it does not present a tripping hazard.**
- ▶ **Never open the device housing. Opening the housing voids the warranty.**
- ▶ **The device must be turned off and the power plug must be removed before it is folded together.**
- ▶ **Carry the device only when it is folded together and by using the carrying handle.**
- ▶ **Only for indoor domestic use. In a medical environment this device may not be operated together with other devices that are used for life support.**
- ▶ **Ensure that other people, especially children and persons who cannot assess this hazard, are aware of this hazard.**

Functional elements

(For illustrations see fold-out pages A1 and A2 left and right)

EN

1. Display
2. Rotary knob to open contrast functions
3. Photo button for saving images and confirming menu options
4. Button for turning the reading line on and off
5. On/Off button
6. Button for opening menu operation
7. Button for overview display
8. Rotary knob to increase or decrease zoom factor
9. USB port
10. SD card slot
11. SD card (insert card as shown)
12. Button for moving upward in menu
13. Button for moving downward in menu
14. Foldable camera cover (flipped open in illustration)
15. Camera
16. Button for moving to right in menu
17. Button for moving to left in menu
18. RESET button for resetting the device
19. Joint for setting screen height and tilt angle
20. Base arm for unfolding the device
21. Charging control LED
22. Connection for power adapter
23. Carrying handle
24. Base plate

Basic operation

Place the device on the base plate 24. Unfold the base arm 20 on the device until it engages (see illustration). The camera 15 is equipped with a cover 14. Open the camera cover and move it to the intended position (see illustration on fold-out pages A1 and A2). If the  symbol is shown, the camera cover has not been opened.

Use the joint 19 to set the required screen height and tilt angle.

Connect the power adapter to a suitable device plug. Connect its cable to the connection for the power adapter 22. Then plug the power adapter into the power socket. The charging control LED 21 lights up while the device is being charged.

To fold the device together again move the screen to the rear. Then fold the camera together and close the camera cover (see illustration). Thereafter, fold the base arm downward.

Turning on/off

Press the On/Off button 5. The camera image will appear on the screen 1 after a few seconds. Symbols briefly show the current status, for example magnification factor and brightness setting.

To turn the device off, press and hold the On/Off button 5 for one second. Current settings are saved.

Reading with the device

Turn the device on and place the reading material on the base plate 24 under the camera 15.

In exceptional cases it can make sense to relocate the field of vision to the upper area of the base plate. To do this, swivel the camera out past the locking position.

Adapting the magnification

To increase the magnification turn the rotary knob ❸ in clockwise direction when the live image is displayed. To decrease the magnification, turn the rotary knob ❸ counterclockwise.

EN

The magnification factor you have set will be displayed briefly.

Turning the reading line/row on and off

Press the ❹ button to turn the reading line on. Press it again to hide the line.

Special button functions

- ▶ Press and hold the  button ❸ to open the photo album.
- ▶ Press and hold the  button ❷ to turn the autofocus function off (see also Chapter "Deactivating/activating autofocus" on page 23).
- ▶ Press and hold the reading line button ❹ to change the reading material brightness (see Chapter "Adjusting the reading material brightness" on Page 27).

False color display

To display the camera image in false colors for increased contrast, turn rotary knob ❷ until the required false color rendering is displayed. By default, the false colors are preset as black/white and white/black. To adapt the false color display, open the "Adjusting false colors" menu (see Page 26).

Saving the live image as a file

Press the  button ❸ briefly to save the current camera image as a file on the inserted SD card and to show it as a still image on the screen. You can also change the magnification factor for this image as already described. You can use the arrow keys ❶ + ❷ to move the magnified image section vertically, and arrow keys ❸ + ❹ to move it horizontally. Press the Photo button again to return to the live image display.

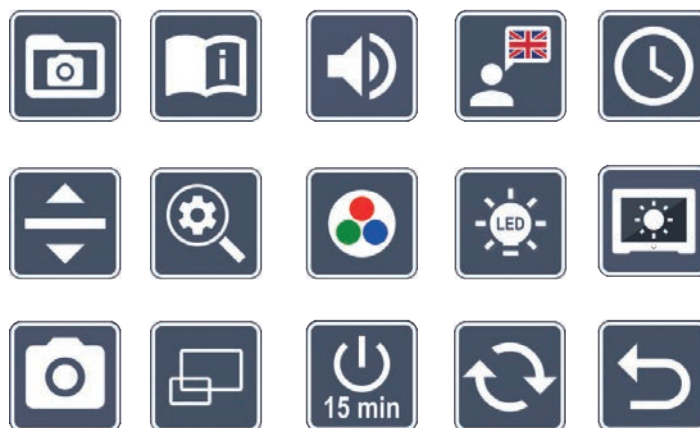
Deactivating/activating autofocus

Press and hold the  button ⑦ to deactivate the autofocus function. This can be helpful, for example while writing under the device, so that the focus remains on the reading material. Press the  button again to reactivate autofocus. Autofocus is automatically reactivated when the device is turned off.

Opening the main menu

Press the MENU button ⑥ to open the menu:

10:36

You can move to the individual symbols using the rotary knobs ② + ⑧ and the arrow keys ⑫, ⑬, ⑯, ⑰. To turn any of the selected functions on or off, press the  button ③.

You can exit the setting menu at any time by simply pressing the MENU button ⑥.

Managing image files on the SD card

Highlight this symbol and then press the  button to open image files from the SD card. The following menu appears:

The currently selected image file will be displayed centrally above it. The previous image is shown to the left of it, the next one to the right.

Use the two rotary knobs ② + ⑧ to select the individual symbols with the following functions:

EN

-  - select the previous image
-  - select the next image
-  - select or deselect the image currently being displayed in the center as a favorite.
-  - the current image is set as a favorite
-  - the current image is not set as a favorite

Tip: the image selected as a favorite will be displayed first when a photo album is opened. Only *one* image can be selected as a favorite at any one time.

-  - delete the selected image from the SD card
-  - back one level

The symbol for the selected function becomes larger and is displayed in color. Press the  button ③ to activate each selected function.

Press the  button ⑦ to display the image file in the middle full screen. You can use the rotary knob ⑧ to set the required zoom factor. You can use the left-hand arrow key ⑯ + ⑰ to move the zoomed-in image section horizontally and the right-hand arrow key ⑫ + ⑬ to move it vertically.

Deleting image files

To delete the selected image file from the SD card, first select the delete symbol  and then confirm with the  button ③. The confirmation symbol  is now shown on the bottom left of the screen, and the cancel symbol  is shown on the right. You can use the rotary knobs ② + ⑧ to select  or . The symbol you have selected will be displayed slightly larger. Press the  button to execute the selected function.

Press the  button ⑦ again to return to the image selection and to show the 5 menu symbols.

Opening the quick guide

Select the national flag that represents the language you desire. Press the  button to change the language. You can use  /  to scroll the page preview backward and forward. Operation is analog to viewing images from the SD card. Press the  button ⑦ to quit full-screen mode.

Adjusting volume

The current volume setting is displayed. Select this symbol and then press the  button ③ to select the following settings:

- very low volume

- low volume

- medium volume

- high volume

- mute

Selecting the language for voice output

Select this symbol and then press the  button ③ to select the language for voice output. The symbol for the current voice output setting is displayed. Alternatively you can select  or  for confirmation using a signal tone.

The  symbol is displayed while the language file is being loaded.

Setting the time

The time will only be displayed in the menu once it has been set. Select this symbol and then press the  button ③ to set the time. Select the  symbol so that you can increase the number of the parameter displayed in white using the  button ③. Select the  symbol so that you can decrease the number of the parameter displayed in white using the  button ③. Select the  symbol and then press the  button ③ to select the next parameter to be set.

Select "24H" and press the  button ③ to change from a 24-hour display to the 12-hour display normally used in English-speaking areas or vice versa.

EN

Adjusting the reading line/row

Select this symbol and then press the  button ③ to set the details for the reading line or row using the following symbols:

- move the reading line position upward or magnify the reading row

- move the reading line position downward or minimize the reading row

- turn the reading line/row on/off

- switch to "red reading line" or "reading row"

- switch between horizontal/vertical reading line/row

Alternatively you can use the rotary knob ⑧ to change the position of the "reading line" or "reading row".

Adjusting maximum magnification

Select the symbol for the maximum magnification factor that should be available.

Adjusting false colors

Select this symbol and then press the  button ③ to select the false color combination that should be available. Black/white plus inverting are activated by default.

Non-activated combinations are shown crossed out.

Select the required combinations and then change their status using the  button ③.

Adjusting the reading material brightness

Select this symbol and then press the  button ③ to set the brightness for the reading material using the following symbols:

- no LED lighting

- the LEDs light up with half power

- the LEDs light up with full power

- turn the LED spots on the screen on or off

Confirm the required brightness using the  button ③.

Adjusting the screen display brightness

Select this symbol and then press the  button ③ to adjust the display brightness using the following symbols:

- display brightness = 50 %

- display brightness = 75 %

- display brightness = 100 %

Turning the photo button on/off

Select this symbol to turn the photo function for the  button ③ on or off. If the photo button has been turned off its symbol will be shown crossed out. If you want to turn the  button on, press the  button again.

Turning the overview display on/off

Select this symbol to turn the overview display function off. If you want to turn the overview display on again, press the  button ③ again.

Adjusting the timeout function

By default, the device turns off automatically after approximately 15 minutes, if it is not being used. Select this symbol and then press the  button ③ to extend this period up to approximately 8 hours.

EN

Updating the firmware

Select this symbol and then press the  button ③ to display the current version of the firmware on the device and to update it if necessary. If you have registered your vario DIGITAL FHD, you will be notified about any available firmware updates.

To update the firmware please proceed as follows:

1. Connect the vario DIGITAL FHD to a PC using the provided USB cable. Ensure that the SD card is inserted in the card slot ⑩.
2. Turn the vario DIGITAL FHD on. The contents of the SD card will now be displayed on the PC as the "ESCHENBACH" drive.
3. Now open the web address that was provided to you in the email about the firmware update using the connected PC.
4. Download the firmware update and save it directly on the drive - not in a drive subfolder.
5. Disconnect the vario DIGITAL FHD from the PC using the "Unplug and eject hardware" function, whose symbol you can find on your operating system taskbar.
6. Open the settings menu, select the  symbol and press the  button.
7. Select the  symbol and press the  button. To cancel the update and return to the previous page, select the  symbol and press the  button.

The following icon will be displayed on the screen during the firmware update:

The vario DIGITAL FHD will turn off automatically once the update is successful.

Never turn the device off and never remove the SD card from the device during the firmware update. Otherwise the vario DIGITAL FHD could be irreparably damaged.

This type of damage is not covered by the warranty!

You can use the  symbol and confirm with the  button ③ to quit the settings menu.

USB port

You can use the supplied USB cable to connect the device to the USB port on a computer using the device USB port ⑨. The SD card inserted in the device will then be displayed on the computer as the "ESCHENBACH" drive for file operations.

Reset

If the device no longer functions as described in these instructions, it may be due to a malfunction, which you can eliminate yourself by resetting the device. Then press the RESET button ⑱.

Technical specifications

Display size:	39.6 cm (15.6") 16:9 Full HD
Camera:	optical/digital zoom camera with AF
Magnification range:	1.3 × - 45 ×
Display modes:	real color, contrasting up to 14 false color combinations possible
Card formats supported:	SD, SDHC (max. 32 GB)
Charger power supply:	Primary: 100 V – 240 V ~ 50 – 60 Hz Secondary: 19 V == 2000 mA
USB port:	Type C (only for data transfer)
Operating temperature range:	+10 °C – +40 °C (50 °F – 104 °F), humidity: < 80 %
Storage temperature range:	0 °C – +60 °C (32 °F – 140 °F), humidity: < 80 %
Dimensions (W × H × D):	378 × 460 × 132 mm (14.9" x 18.1" x 1.6")
Weight:	~ 6.3 kg (13.9 lbs)
Lighting:	LED
Power consumption:	~ 16 W
Images per second.:	60 fps
Contrast ratio:	1000 : 1

These instructions are also available on the Internet at <http://www.eschenbach-optik.com/>

USB® is a registered trademark of USB Implementers Forum, Inc.

Care instructions

- ▶ Turn off the device and disconnect it from the power source before you clean it.
- ▶ Clean the device using a soft, dry cloth only.
- ▶ Never use soap solutions containing plasticizers or alcoholic or organic solvents to clean your device, and never use abrasive detergents.
- ▶ Never clean the device in an ultrasonic bath or under running water.
- ▶ Protect your device against heavy contamination.
- ▶ Condensate may form and cloud the lens when you move from cold rooms to warm rooms. If this happens, wait 40 minutes before starting the device to avoid condensate formation.

EN

Disposal

- ▶ This product is subject to the provisions of European WEEE Directive covering old electrical and electronic devices in its currently applicable version. Never dispose of your old device in normal domestic waste. Instead dispose of it through an approved disposal facility or your municipal waste facility. Observe the regulations applicable in your country. In case of doubt, contact your waste disposal facility. Dispose of all packaging materials in an environmentally friendly manner.

Warranty

Within the parameters of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing defects or material defects. No warranty claims can be accepted for damage that arises due to improper handling, including damage from falls or impacts. Warranty claims can only be accepted on presentation of the sale receipt.

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USB® is een wettig gedeponiseerd handelsmerk van USB Implementers Forum, Inc.